

Attends

Attends Pull-Ons Fitting Guide



1 Pull-Ons

Attends Pull-Ons can be used like ordinary underwear and the pad can be pulled up and down as many times as you like.



2 Fit

The elasticity of the pants ensure a snug, comfortable fit whilst the breathable material is skin friendly.



3 Removal

Changing and removing the pad is very easy: either pull the pant down or you can tear open both sides of the pants.

Hints and Tips

Selection

- Choose the size of Pull-On you require according to hip and waist size – whichever is larger
- Choose the absorbency you require based on the volume of urine passed.
- Pull-Ons can offer effective management for people who may pull out shaped pads

Disposal

- Roll up pads and dispose as per your trust's clinical waste policy
- Do not flush the pad down the toilet

For further information visit our website on www.attends.co.uk