

# Attends

## Pull-Ons

|   |   |   |   |   |   |   |   |   |   |    |
|---|---|---|---|---|---|---|---|---|---|----|
| 0 | 1 | 2 | 3 | 4 | 5 | 6 | 7 | 8 | 9 | 10 |
|---|---|---|---|---|---|---|---|---|---|----|

*The Attends numbering system – Absorption degree*

**Question:**

**When does the bladder weakness occur?**

**What happens?**

Attends Pull-Ons is a range of highly absorbent pants that can be pulled up and down by the client or the caregiver. They are available in 2 absorption levels: Plus (5) and Super (8) and can manage both urinary and faecal incontinence. The pants are white for discretion.

Due to their flexible shape with asymmetric leg-cut, Attends Pull-Ons give the feeling of normal underwear.

Attends Pull-Ons have a highly absorbent core and anti-leakage barriers, which help to prevent leakage. The urine is kept away from the skin and the odour protection core locks unpleasant smells inside the product.

A wide elastic panel in the waistband (both front and back) and soft leg elastics provide optimum comfort for the wearer. The cloth-like backsheet of Attends Pull-Ons is made of soft, breathable material, which contributes to healthy skin. If needed, the side panel can easily be torn open, for quick disposal.

Attends Pull-Ons are available in a range of sizes.

**Answer:**

The client has reduced bladder and /or bowel sphincter control and incontinence can occur without the client being aware.

Loss of urine and sometimes also faeces, several times during the day and night.



# Attends

## Pull-Ons

|   |   |   |   |   |   |   |   |   |   |    |
|---|---|---|---|---|---|---|---|---|---|----|
| 0 | 1 | 2 | 3 | 4 | 5 | 6 | 7 | 8 | 9 | 10 |
|---|---|---|---|---|---|---|---|---|---|----|

The Attends numbering system – Absorption degree



| Size                |       | XX-Small | X-small  | Small    | Medium   | Large    | X-Large  | Medium   | Large    |
|---------------------|-------|----------|----------|----------|----------|----------|----------|----------|----------|
| EAN Case            |       | 20 38 97 | 20 10 91 | 20 11 07 | 20 06 98 | 20 07 11 | 20 37 36 | 20 05 99 | 20 06 12 |
| Waist size          | cm    | 38-55    | 45-60    | 56-75    | 75-110   | 100-160  | 120-175  | 75-110   | 100-160  |
| Leg opening         | cm    | 35-46    | 38-50    | 40-58    | 45-78    | 56-90    | 60-97    | 45-78    | 56-90    |
| Weight              | g     | 71       | 64       | 64       | 75       | 100      | 100      | 101      | 106      |
| Rise (crotch-waist) | cm    | 30       | 34       | 34       | 41       | 41       | 45       | 41       | 41       |
| Abs. (ISO- 11948-1) | ml    | 1520     | 1400     | 1400     | 1640     | 2200     | 2070     | 2440     | 2440     |
| Odour Protect.      |       | Yes      | Yes      | Yes      | Yes      | Yes      | Yes      | Yes      | Yes      |
| Bag count           | pcs.  | 21       | 21       | 21       | 14       | 14       | 14       | 14       | 14       |
| Case count          | bags. | 4        | 4        | 4        | 4        | 4        | 4        | 4        | 4        |
| Pallet height       | cm    | 179      | 200      | 200      | 185      | 185      | 175      | 185      | 185      |
| Cases/Layer         |       | 4        | 4        | 4        | 5        | 5        | 5        | 5        | 5        |
| Layers/Pallet       |       | 6        | 5        | 5        | 5        | 5        | 5        | 5        | 5        |
| Products/Pallet     | pcs   | 2016     | 1680     | 1680     | 1400     | 1400     | 1400     | 1400     | 1400     |