

Hints and Tips

Selection

- Think about using Attends pad / pants before using Slips – they offer much greater discretion
- Attends Stretch Pants should be worn with Contours 6 and above
- Choose the Slip according to hip or waist size – whichever is the largest
- Choose the absorbency based on the volume of urine passed
- Slips can offer effective management for people who may pull out shaped pads

Fitting

- Ensure you keep the ruffles on the flexible leg gathers on the outside of the Slip – this keeps the waterproof back sheet away from the skin
- Keep the adhesive tapes at the back when fitting the Slip – you then know it is the right way round
- Let the anatomy of the user decide whether the wing is applied like a belt or positioned in a more downwards direction
- If the wings overlap the tapes can be fixed on top of the wing
- If the product is fitted when the wearer is lying down, to make sure the pad stays in place, place a hand on the back of the Slip when the user is turned from their side on to their back
- Use the wetness indicator to centre the whole pad

Changing

- Check the wetness indicator – if the wetness indicator has disappeared over half the length of the pad, the product requires changing. If not the pad can be reapplied
- Reseal and reuse the Slip when toileting if the product is clean and dry
- When undoing the tapes the blue tape should remain in place on the product
- Do not stick tapes to the skin
- Remove soiled pads from the rear where possible
- Clean skin as per your local policy. Clean from front to back
- Avoid oil based creams if possible and powders should not be used at all as these will affect the absorbency of the pad, preventing urine being absorbed and placing the skin at risk

Leakage

- Check fitting
- Check sizing
- Only increase the absorbency of the pad when you are happy it is not a fitting issue and following clinical assessment

Disposal

- Roll up pads and dispose as per your trust's clinical waste policy
- Do not flush the pad down the toilet

For further information visit our website on www.attends.co.uk

Attends

Attends Slips Fitting Guide

Standing



Preparation

Measure the hips and waist and using the larger of these two measurements ensure that the correct product size is selected.



Preparation

Open the product fully and position the end with the tapes to the back ensuring that the wings are fully opened before applying the pad. Make sure the waterproof backing of the Slip is on the outside and the soft white inner lining is next to the skin.



Fitting

Fold and cup the pad and position the pad between the legs so that it fits snugly to the wearer and that it fits comfortably in the groin area. Use the wetness indicator to help to position and centre the pad.



Fitting

Gently pull the wing around to the front of the pad. Use the blue positioning tape to pre-apply the product on both sides.




Fitting

Readjust as necessary to get the best fit:

- 1 Pull up the Slip to make sure it is closely fitted to the body especially in the groin area.
- 2 Adjust the fit.

**Fitting**

Fasten the tapes firmly onto the product.
When you remove the Slip lift the white tape off the blue tape.

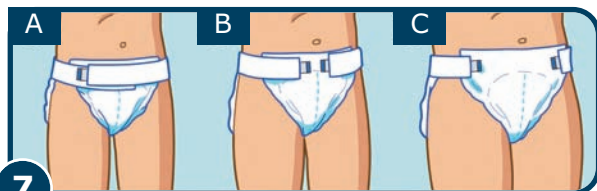**Fitting**

Depending on the anatomy of the user the wings may

- A Overlap
- B Be close to each other
- C Be close to the edge of the Slip

Laying Down



1

Preparation

Measure the hips and waist and using the larger of these two measurements ensure that the correct product size is selected.



2

Preparation

Open the product fully and position the end with the tapes to the back ensuring that the wings are fully opened before applying the pad. Make sure the waterproof backing of the Slip is on the outside and the soft white inner lining is next to the skin.



3

Fitting

Fold and cup the pad and position the pad between the legs so that it fits snugly to the wearer and that it fits comfortably in the groin area. Use the wetness indicator to help to position and centre the pad.



4

Fitting


Gently pull the wing around to the front of the pad. Use the blue positioning tape to pre-apply the product on both sides and readjust as necessary to get the best fit.



5

Fitting

Fasten the tapes firmly onto the product.
When you remove the Slip lift the white tape off the blue tape.

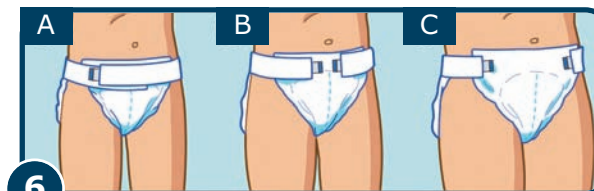

6

Fitting

Depending on the anatomy of the user the wings may

- A Overlap
- B Be close to each other
- C Be close to the edge