

# Attends

## Stretch Pant

|   |   |   |   |   |   |   |   |   |   |    |
|---|---|---|---|---|---|---|---|---|---|----|
| 0 | 1 | 2 | 3 | 4 | 5 | 6 | 7 | 8 | 9 | 10 |
|---|---|---|---|---|---|---|---|---|---|----|

The Attends numbering system – Absorption degree

### Question:

**When do I use Attends Stretch Pants?**

### Answer:

Attends Stretch Pants hold Attends Contours 5 to 10 in place. They can also be used with other Attends pads, like Discrete 3 and 4 if close fitting underwear is not available. It is important to keep the pad close to the body, to ensure fast acquisition and reduce the risk of leakage.

### What makes Attends Stretch Pants so special?

Attends Stretch Pants is the cost effective option from Attends pants and are available in 5 different sizes.

**Attends Stretch Pants** are pants with short legs designed to be used in conjunction with Attends Contours or Rectangulars to provide optimum comfort and security and improved product performance. The pant is made of a soft stretchy material that gives a perfect fit. The pants are available in 5 different sizes: Small, Medium, Large, Extra Large and Extra Extra Large and are suitable for both men and women. A coloured strip on the pant helps to determine the correct size of pant. All Attends pants are white for extra discretion.

Washing Instructions: Washable up to 15 times in 40-85°C.



|                          |        |                    |                    |                    |                    |                    |
|--------------------------|--------|--------------------|--------------------|--------------------|--------------------|--------------------|
| Art. No (case)           |        | 428 051            | 703 684            | 503 222            | 328 016            | 203 088            |
| Waist size               | cm     | 40-80              | 70-100             | 90-120             | 110-135            | 130+               |
| Patients weight          | kg     | 20-40              | 40-75              | 75-90              | 90-110             | 110+               |
| Coloured strip           |        | Red                | Blue               | Brown              | Green              | Orange             |
| Materials                |        | Polyamide Elastane | Polyamide Elastane | Polyamide Elastane | Polyamide Elastane | Polyamide Elastane |
| Packcount                | Pcs    | 15                 | 15                 | 15                 | 15                 | 15                 |
| Cascount                 | Packs  | 12                 | 12                 | 12                 | 10                 | 10                 |
| No. of cases per layer   | Cases  | 8                  | 8                  | 8                  | 8                  | 8                  |
| No. of layers per pallet | Layers | 8                  | 8                  | 8                  | 8                  | 8                  |