


Attends® Soft




Attends - The system with 10 absorption levels for bladder weakness

Attends Soft are a range of small anatomically shaped pads designed to manage very light to heavy urinary incontinence.

Soft 0-3+

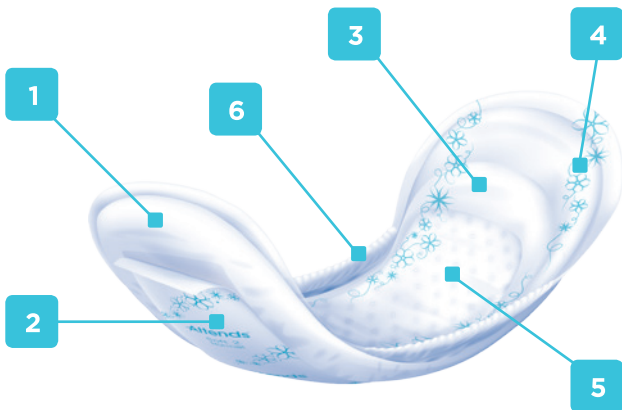

Soft 4-7

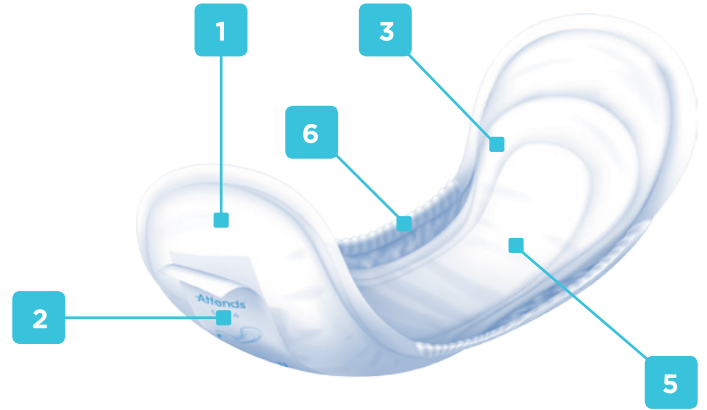

- 1** White breathable back sheet to aid containment and for improved comfort, discretion and skin health.
- 2** Fixation tapes allows the pads to be worn with close fitting underwear for extra discretion.
- 3** Anatomically shaped core for a close comfortable fit.
- 4** Super Soft printed top sheet (Soft 0 - 3+) for improved softness and a more aesthetic look with discreet blue printing.
- 5** Superabsorbent core with acquisition layer for improved leakage protection, skin dryness and odour protection.
- 6** Soft foam elastics (Soft 2 - 7) helps to provide a natural cup shape for improved fit and to reduce the risk of leakage.



Attends Soft is tested and approved by proDERM, Institute for Applied Dermatological Research.

For more information visit www.attends.co.uk

Attends Soft

	Packcount Casecount	EAN 7332152 Pack / Case	Product Length (cm) Product Width in Crotch (cm)	Backsheet	Absorption Core Material	Topsheet Material	Elastics	Fixation Tapes	Natural Odour Protection	Breathable	Quick-Dry Technology
Attends Soft Micro 0	28 8	205266 205273	17.5 6	Laminate of nonwoven and breathable film	Airlaid core of fluff, superabsorbents and synthetic fibres with Quick-Dry acquisition layer	Non-woven	X	✓	✓	✓	✓
Attends Soft Ultra Mini 0	28 8	205068 205075	20.5 6.5	Laminate of nonwoven and breathable film	Airlaid core of fluff, superabsorbents and synthetic fibres with Quick-Dry acquisition layer	Non-woven	X	✓	✓	✓	✓
Attends Soft Mini 1	20 12	204542 205327	21.5 8	Laminate of nonwoven and breathable film	Fluff and superabsorbents with Quick-Dry acquisition layer	Non-woven	X	✓	✓	✓	✓
Attends Soft Mini Long 1	16 12	204566 204863	27.5 8	Laminate of nonwoven and breathable film	Fluff and superabsorbents with Quick-Dry acquisition layer	Non-woven	X	✓	✓	✓	✓
Attends Soft Normal 2	12 8	207239 207246	26.5 10.5	Laminate of nonwoven and breathable film	Dual Core system with fluff and superabsorbents with Quick-Dry acquisition layer	Non-woven	Foam elastic	✓	✓	✓	✓
Attends Soft Extra 3	10 8	207253 207260	32.5 12	Laminate of nonwoven and breathable film	Dual Core system with fluff and superabsorbents with Quick-Dry acquisition layer	Non-woven	Foam elastic	✓	✓	✓	✓
Attends Soft Extra Plus 3+	10 12	204665 205112	36.5 13	Laminate of nonwoven and breathable film	Dual Core system with fluff and superabsorbents with Quick-Dry acquisition layer	Non-woven	Foam elastic	✓	✓	✓	✓
Attends Soft Super 4	46 3	204740 207673	38.5 14.5	Laminate of nonwoven and breathable film	Dual Core system with fluff and superabsorbents with Quick-Dry acquisition layer	Non-woven	Foam elastic	✓	✓	✓	✓
Attends Soft 5	38 4	204764 204771	47.5 16.5	Laminate of nonwoven and breathable film	Dual Core system with fluff and superabsorbents with Quick-Dry acquisition layer	Non-woven	Foam elastic	✓	✓	✓	✓
Attends Soft 6	38 4	204788 204795	47.5 16.5	Laminate of nonwoven and breathable film	Dual Core system with fluff and superabsorbents with Quick-Dry acquisition layer	Non-woven	Foam elastic	✓	✓	✓	✓
Attends Soft 7	34 4	204801 204818	47.5 16.5	Laminate of nonwoven and breathable film	Dual Core system with fluff and superabsorbents with Quick-Dry acquisition layer	Non-woven	Foam elastic	✓	✓	✓	✓



Quick-Dry

Quick-Dry is the acquisition layer placed directly below the top sheet where the urine enters the pad. It quickly absorbs urine away from the skin and into the pad and prevents the urine returning to the surface. It can handle several gushes of urine without affecting how the pad works which results in exceptional skin dryness and leakage protection for the user.



Natural Odour Protection

Natural Odour Protection reduces the risk of unpleasant smells.



Active Zone

The Active Zone is an area centred in the middle of the pad, which provides high absorption where it is really needed. This provides exceptional skin dryness and improved leakage protection for the user.



Super Soft Top Sheet

Softer, skin friendly top sheet.



pH Value

The core technology with the curly fibre acquisition layer helps to guarantee a low pH which is similar to the skin. This helps to protect the skin and ensures improved skin health.



Breathable

The breathable back sheet enables the skin to breathe, which contributes to improved skin health.



Single Wrapped

The products are single wrapped for improved hygiene and discretion.