


Attends® Pull-Ons



Fitting Guide


1

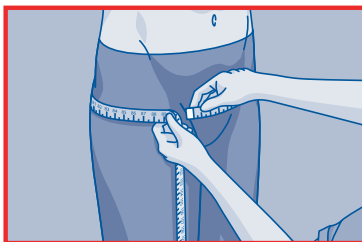

Preparation

Measure the hip and waist and using the larger of these two sizes ensure that the correct product size is selected.


2

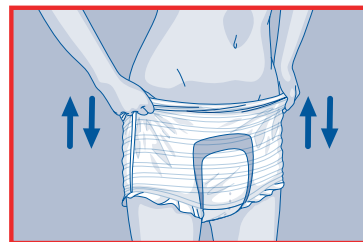

Fitting

Attends Pull-Ons can be used like ordinary underwear and the pad can be pulled up and down as many times as you like.


3

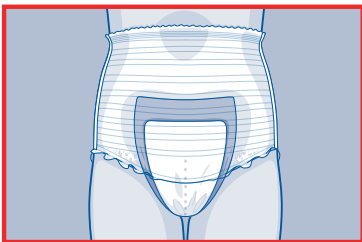

Fit

The elasticity of the pants ensure a snug, comfortable fit whilst the breathable material is skin friendly.

4

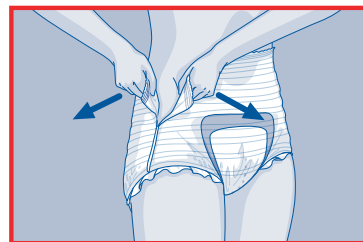

Removal

Changing and removing the pad is very easy: either pull the pant down or you can tear open both sides of the pant.

Hints and Tips

Selection

- Choose the size of Pull-On you require according to hip and waist size - whichever is larger.
- Choose the absorbency level based on the volume of urine passed. Start by choosing a lower absorbency number for example level 3 before moving to a higher absorbency for a discreet and more comfortable fit.
- Pull-Ons can offer effective management for people who may pull out shaped pads.

Fitting

- Ensure that the product is fitted the correct way round.

Changing

- Check the wetness indicator - if the wetness indicator has disappeared over half the length of the pad (60%) the product requires changing. If not the pad can be reapplied.

Disposal

- Roll up pads and dispose as per your Trust's clinical waste policy.
- Do not flush the pad down the toilet.

For more information visit www.attends.co.uk